



ITEM NO.	PART NUMBER	DESCRIPTION	QTY.
1	0871910045 Pedistirrup Trolley	Stel	1
2	SW0001004	skiver	2
3	Ø100 tente Hjul	Ø100 Hjul	4
4	SW0002676	M8 x 40 cyl.	4
5	SW0000999	M4 x 12 mm unbr.	8
6	User Library-allen wrench	Nøglesæt	1
7	SW0001000	M8 x 10 pinolskruer	8
8	SW0001003	M8 x 40 cyl.	2
9	SW0002678	Samlevejledning	1
10	Closure profile 20x40	Ende prop 20x40	4
11	Closure profile 25x50	endeprop 25x50	2
12	SW0001001	M6 x 10 Pinolskrure	2


Kunde:		Ordre.nr	Vægt (kg.)	31355.80
Matr type:	A3	Dato:	08-01-2021	Konstruktør:
Pladefykkelse:		Rev.Dato:		
Matr:	SCALE 1:8	Overflade:	SHEET 1 OF 2	
Tegningsbeskrivelse:				Tegningsnr:
				87855-00

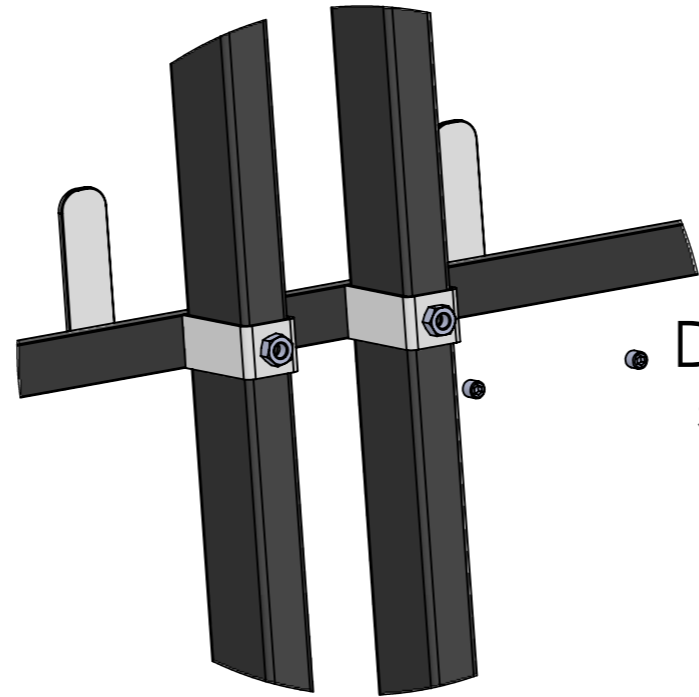


● DETAIL A
SCALE 2:7

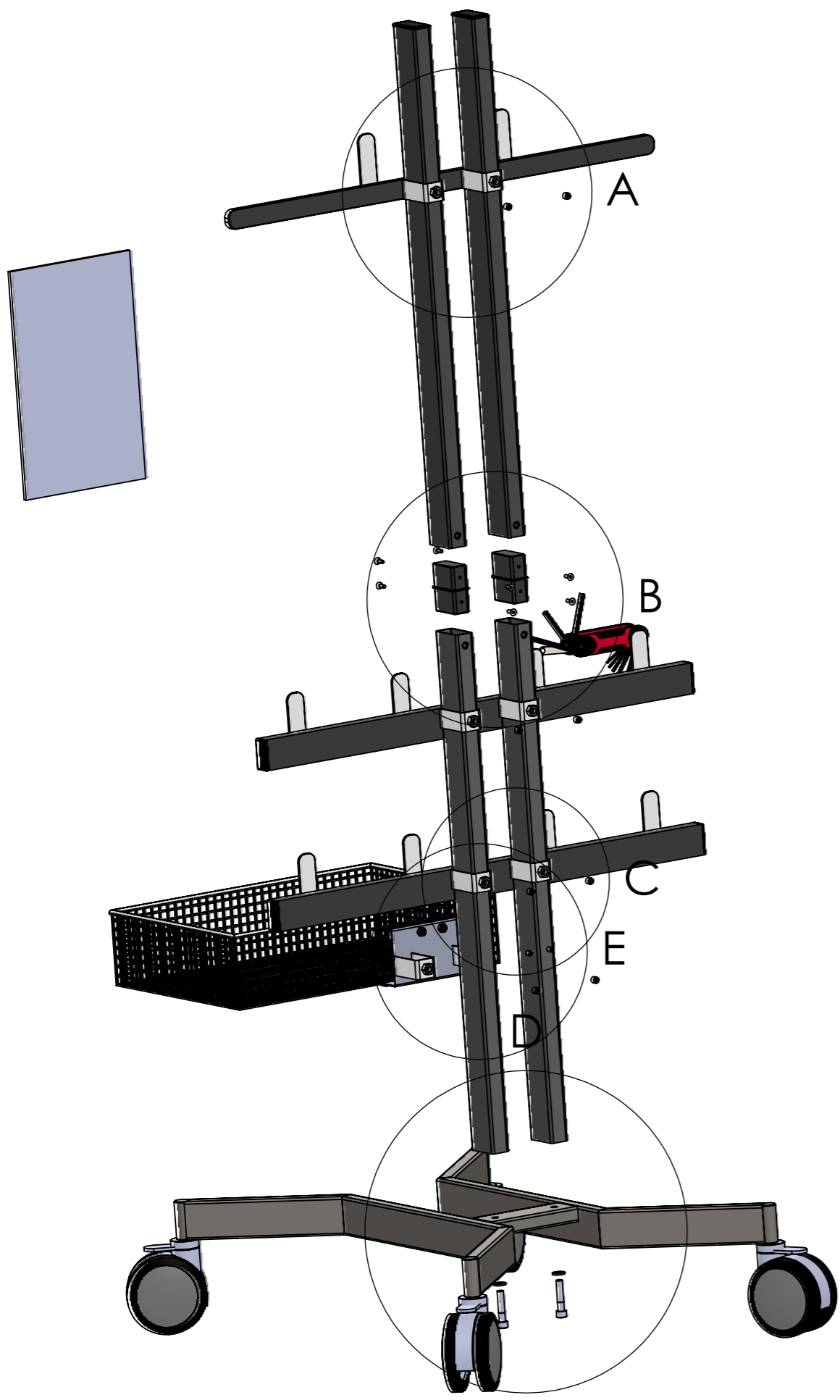


● DETAIL B
SCALE 2:7

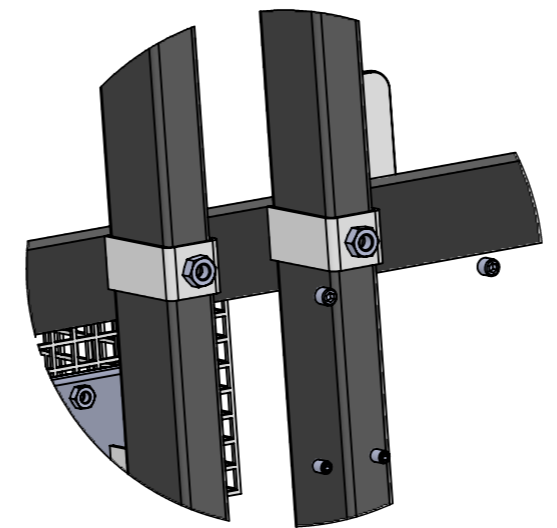

● DETAIL C
SCALE 2:7

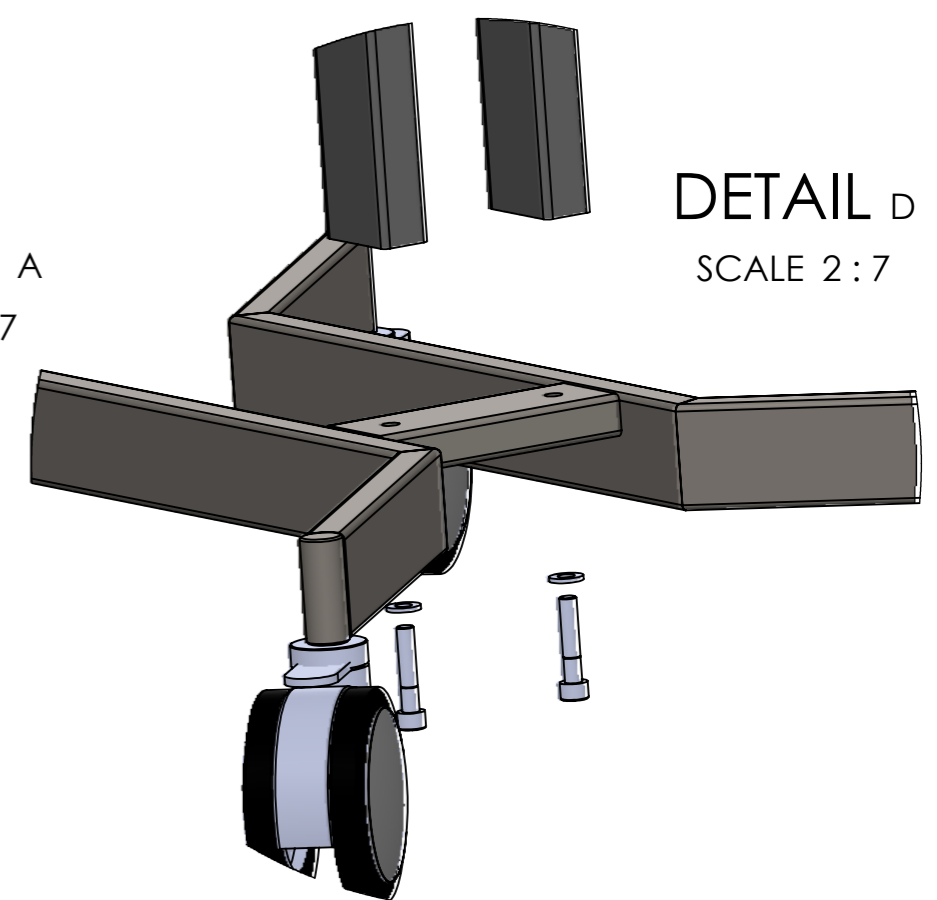

● DETAIL D
SCALE 2:7

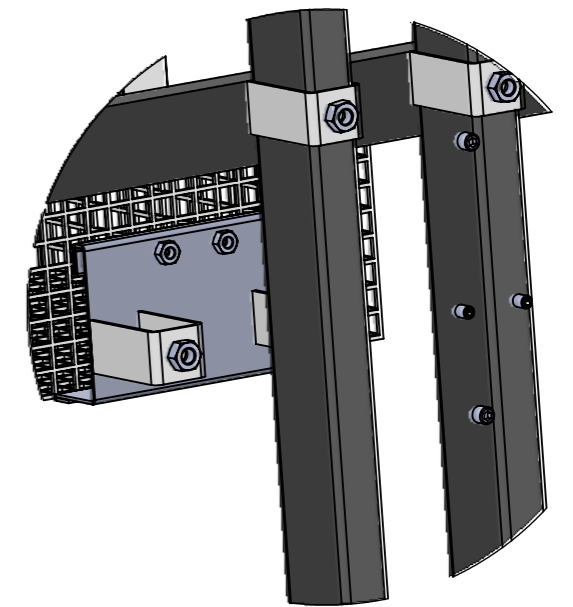

Kunde:	Ordre.nr	Vægt (kg.)	31355.80
Matr type:	A3	Dato:	19.11.2021
Pladetykkelse:	SCALE 1:7	Rev.Dato:	
Matr:		Overflade:	SCALE 2:7
Tegningsbeskrivelse:		Tegningsnr:	87855-00